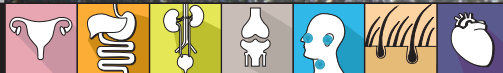
TONTARRA

ENDOSCOPY

PLASMA WAVE HF GENERATOR PWT-400

CLEVER. SAFE. POWERFUL

PLASMA WAVE HF GENERATOR

Many options – one device.

Easy handling.

► **CLEVER**

► **SAFE**

► **POWERFUL**

CLEVER

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Twin foot-switch
(Art. No. 540-285-50)

HF-Verbindungskabel PWT-400

HF-connection cable PWT-400
International Models; 2-pins; 3.0 m
(Art. No. 287-215-30BR)

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For further accessories please have a look into our HF-accessories catalogue.

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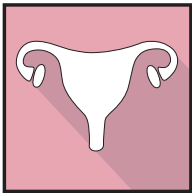
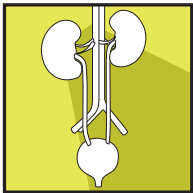
Gynecology, urology, general and thoracoscopic surgery, orthopaedics, maxillofacial surgery, dermatology, plastic surgery, vascular surgery, paediatric surgery, emergency surgery, gastroenterology, veterinary... and other.

INDIKATIONEN / INDICATIONS

		
▶ Laparoskopische suprazervikale Hysterektomie (LASH) ▶ Laparoskopisch assistierte vaginale Hysterektomie (LAVH) ▶ Totale laparoskopische Hysterektomie (TLH) ▶ Vaginale Hysterektomie ▶ Laparoscopic supracervical hysterectomy (LASH) ▶ Laparoscopically assisted vaginal hysterectomy (LAVH) ▶ Total laparoscopic hysterectomy (TLH) ▶ Vaginal hysterectomy	▶ Transurethrale Resektion (TUR-B, TUR-P) ▶ Zystektomie ▶ Nephrektomie ▶ Transurethral resection (TUR-B, TUR-P) ▶ Cystectomy ▶ Nephrectomy	▶ Kolektomie ▶ Fundoplikatio ▶ Gastrektomie ▶ Splenektomie ▶ Cholezystektomie ▶ Leberresektion ▶ Colectomy ▶ Fundoplication ▶ Gastrectomy ▶ Splenectomy ▶ Cholecystectomy ▶ Liver resection

Our own instrument systems match a wide range of options for diagnostics and therapy in endoscopic-urology (cystoscopy and laser-cystoscopy, resectoscopy, urethrotomy, uretero-rensoscopy, nephroscopy and instruments for pediatric urology).

Contact us for further information about

- the compatabilty of your imaging system and your instruments to our telescopes,
- the imaging solutions and devices we can provide for urology
- our innovative endoscopic instruments and accessories,
- ... and our „uro portfolio“ in general: info@tontarra.de

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TONTARRA'S PORTFOLIO INCLUDES

GUBBINI – A NEW DIMENSION IN HYSTERO-RESECTOSCOPY

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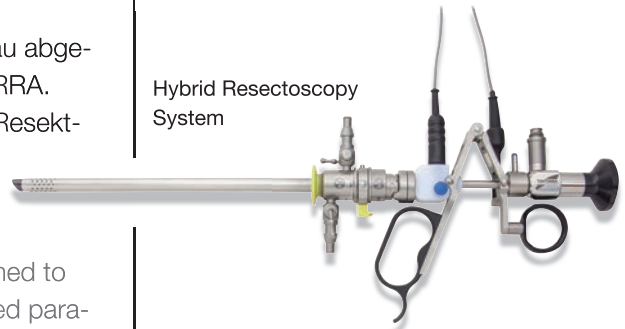
GUBBINIoriginal

The original GUBBINI System features a 16 Fr. round distal tip/sheath form, the introduction into the cervical canal does not require dilation.

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
Hybrid Resectoscopy System

ENDOOR C – YOUR KEY TO PERFECT LAPAROSCOPIC PERFORMANCE

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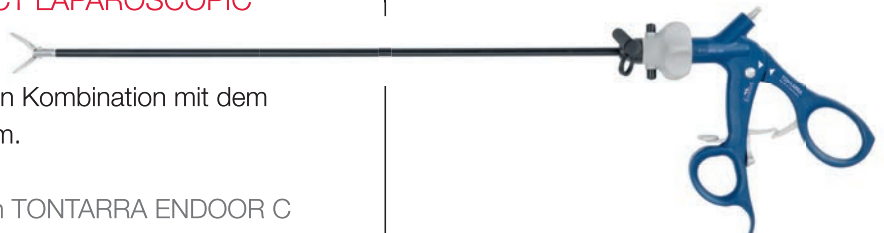BIPOLARE KOAGULATION / BIPOLAR CUTTING MODES

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	Fulg forced coag	Spray coag	Pin point contact coag	pulsed spray coag	Soft micro coag
Max. output power	150 W	200 W	300 W	200 W	200 W
Rated load	300 Ω	700 Ω	400 Ω	700 Ω	300 Ω
Vpp	4500	7750	3700	7850	3300
Crest factor	6.45	7.75	2.2	11.54	2.16
Modulation	60 kHz	30 kHz	17 kHz	3 Hz	17 kHz
Duty Cycle	18%	7%	85%	50%	75%
Power setting	Progressive with steps from 1 to 10 W				10 effects
Power control	ADC System – dynamic self-adjusting with constant voltage				APC System – totally self-adjusting with constant voltage

	Saline Uro-Gyn cut	Saline Uro-Gyn cut	Standard bicut	Blend bicut
Max. output power	350 W	230 W	200 W	160 W
Rated load	350 Ω	350 Ω	300 Ω	200 Ω
Vpp	1070	1070	1070	740
Crest factor	1.5	1.5	1.5	1.57
Modulation	0	1.5 kHz	0	17 kHz
Duty Cycle	100%	100%	100%	95%
Power setting	10 effects		Progressive with steps from 0.1 to 5 W	
Power control	ADC System – dynamic self-adjusting with constant voltage			

	Soft micro bicoag	Auto soft micro bicoag	Sealing	Auto sealing	Forced macro bicoag
Max. output power	140 W	140 W	320 W	320 W	200 W
Rated load	100 Ω	100 Ω	50 Ω	50 Ω	100 Ω
Vpp	460	460	780	780	550
Crest factor	1.56	1.56	2.47	2.47	1.6
Modulation	0	0	0	0	0
Duty Cycle	100%	100%	100%	100%	100%
Power setting	Progressive with steps from 0.1 to 5 W		10 effects		Progressive with steps from 0.1 to 5 W
Power control	APC System – totally self-adjusting with constant voltage			ADC System – dynamic self-adjusting with constant voltage	

Mains supply and consumption, Mains switch, Fuses and Standby	100 / 230V $\pm$ 10% ~ 50/60Hz (Consumption: Max. 3.6 A = 828 VA, Standby 0.4 A = 92 VA). Bipolar switch (green), Two fuses 8 A, Standby mode selectable on the touch-screen.
Equipotential connection	Standard DIN 42801 plug.
Enclosure, Cooling	Metal painted case, protected against the spilling of liquids according to IEC 60601-2-2. Cooling by convection (No fan).
Software updating and calibration	Updating by a serial port connected to a PC, calibration on-site.
Dimensions and weight	38 x 35 x 19 cm (10 kg)
HF generator	Meets IEC 60601-1 and IEC 60601-2-2
EMC compatibility	Meets IEC 60601-1-2
EC Classification and approval	Class IIB according to 93/42/CEE
Classification according to IEC 60601-1	Class I, CF type with the following typical low frequency leakage currents (all cables tied together): on the patient: 4 μ A = 0,004 mA/into the enclosure: 1 μ A = 0,001 mA /to earth: 30 μ A = 0,03 mA
Output circuit according to IEC 60601-2-2	FLOATING (Isolated at high and low frequencies) and protected against the use of the defibrillator. Typical HF leakage currents measured at the unit: monopolar currents = 90 mA, bipolar currents = 60 mA.
Working frequency	Monopolar and bipolar currents: 440kHz $\pm$ 5 w%.

540-280-50	HF-Gerät für Elektrochirurgie PWT-400/HF generator for electrical surgery PWT-400
540-285-50	Doppelfußpedal/Twin foot pedal

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www.tontarra.de

All rights are reserved. Technical alterations are possible. This folder may be used for no other purpose than offering, buying and selling of our products. No part may be copied or reproduced in any form. In the case of misuse, we retain the rights to recall our catalogs and pricelists and to take legal actions.
999-303-01DE · 11.2017

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TONTARRA Medizintechnik GmbH