



> A new dimension in
water-jet surgery.

Precise, selective tissue separation.

An innovation in
tissue-selective
separation/dissection.

The ERBEJET 2 dissects sharply and precisely with minimal trauma to fibrin-rich structures such as vessels, nerves and ducts. Surgeons have commented on the enhanced anatomic visibility, uncovering structures with the ERBEJET 2 not commonly identified using other techniques – including blunt dissection. The ultimate in precision and tissue-selectivity, they often use the ERBEJET 2 to define anatomy and dissection planes prior to transection.

This device produces no heat, making it the ultimate tool for difficult dissection.



ERBEJETTM 2

An innovation in tissue preservation.

The extremely thin laminar jet, rotated in a helical fashion, forces softer, more water-soluble tissue to separate, while fibrin-rich structures are spared. This optimizes the preservation of vessels, ducts and nerves. The preservation of structures is important where cutting of vessels is common, such as hepatic (liver) resection. The potential for blood loss is minimized due to the unique vessel-sparing capability during hepatobiliary surgery.*

ERBEJET 2 also offers a benefit in applications where nerves are particularly at risk, such as during nerve-sparing radical retropubic prostatectomy and total mesorectal excision.

This water doesn't burn.


ERBEJET 2 utilizes precise, gentle pressure, rather than intense heat. This drastically reduces unintentional destruction of healthy tissue, accidental burns, and poor visualization, all of which are common side effects of thermal/ultrasonic modalities.

See the clear advantages.

ERBEJET 2 separates water-soluble tissue while minimizing the unintentional destruction of fibrin-rich structures. The precision and visibility of the geometric stream allow blood vessels, nerves and ductal structures to remain intact.



Tissue-selective dissection

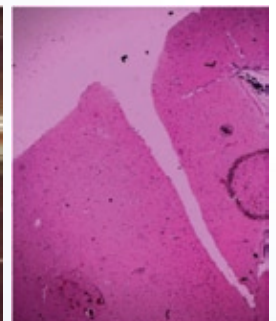

Sharp, precise dissection

A wide variety of open and laparoscopic applicators are available.

Fully Integrated Workstation

- VIO® System – Electrosurgery
“Full ESU Function”
- APC 2 – Argon Plasma Coagulation
- BiClamp® – Vessel Fusion*
- ERBEJET™ 2 – Water-Jet Dissection

* Comparable to other vessel sealing devices.



{ ERBEJET 2 ADVANTAGES }

- > *Selectively dissects, sparing vessels, nerves and ducts*
- > *Potential to minimize blood loss**
- > *Enables better visualization of vital structures*
- > *Reduces trauma to surrounding tissue*
- > *Used in both open and laparoscopic procedures*
- > *Precisely, accurately targets tissue for dissection*
- > *No thermal damage*
- > *Minimized hemorrhage*
- > *Space-efficient design can be ceiling-mounted*
- > *Compatible with ERBE VIO® systems and APC 2*

*Clinical data available upon request





DESCRIPTION

PART NUMBER

ERBEJET 2	10150-000
ESM 2 Suction Module	10340-000
ERBEJET 2 One Pedal Footswitch with ReMode	20150-102
VIO Cart w/Footswitch Holder, Cable Wrap & 1 Bottle Fixation	20180-000
Fastening Set for VIO Cart to Attach VIO/APC 2/VEM 2/ERBEJET 2	20180-131
Fastening Set for VIO Cart to Attach EIP 2/ESM 2/VEM 2	20180-135
Side Rail, Length 390 mm (for VIO Cart)	20180-042
Side Rail, Length 260 mm (for VIO Cart)	20180-040
Hook to Hang Irrigation Bag on Side Rail of Cart	20325-002
Silicone Tubing, 60 cm	077.0922
Suction Jar, Polycarbonate, 2 ltr	077.0221
Lid for Polycarbonate Canister (Suction Jar) for Use w/Disp. Suction Bags	077.0410
Clampholder, Plastic (for ESM 2 Suction Container)	077.0521

DISPOSABLE ACCESSORIES:

ERBEJET 2 Pump Cartridge (Box of 5)	20150-300
ERBEJET 2 Probe, Flexible, O.D. 1.3 mm x Length 2.2 m (Box of 5)	20150-220
ERBEJET 2 Applicator, Blunt Dissector, O.D. 5 mm x Length 336 mm Curved Tip (Box of 5)	20150-226
ERBEJET 2 Applicator w/Suction, Length 65 mm Flexible Sheath/Rigid Tip (Box of 5)	20150-230
ERBEJET 2 Applicator w/Suction O.D. 6 mm x Length 306 mm Rigid Sheath and Tip (Box of 5)	20150-238
ERBEJET 2 Applicator w/Suction O.D. 2.8 mm x Length 90 mm Bayonet Sheath/Tip (Box of 5)	20150-241
ERBEJET 2 Applicator, Blunt Dissector, O.D. 5 mm x Length 183 mm Curved Tip (Box of 5)	20150-225
ESM 2 Suction Bag, 2 ltr (Box of 25)	077.0320
ESM 2 Overflow Protection/Bacteria Filter w/Medela Connections (Box of 10)	077.0571

ERBE
USA INCORPORATED
Surgical Systems

2225 Northwest Parkway
Marietta, GA 30067 USA
www.erbe-usa.com
EMAIL sales@erbe-usa.com

CUSTOMER SERVICE
800.778.3723